

stryker[®]

Neuro Spine ENT

SILVERGlide[®] Non-Stick Bipolar Forceps




SILVERGlide® Bipolar Forceps

SILVERGlide® Keyhole™ Maximum Access

Bipolar Forceps For Minimally Invasive Procedures

- Maximum Access:** choose the length of tine and tip style to meet your surgical needs.
- Maximum Visibility:** micro-gauge tines keep the tissue in view and not the forceps.
- Maximum Performance:** with proprietary Silverglide® non-stick technology.

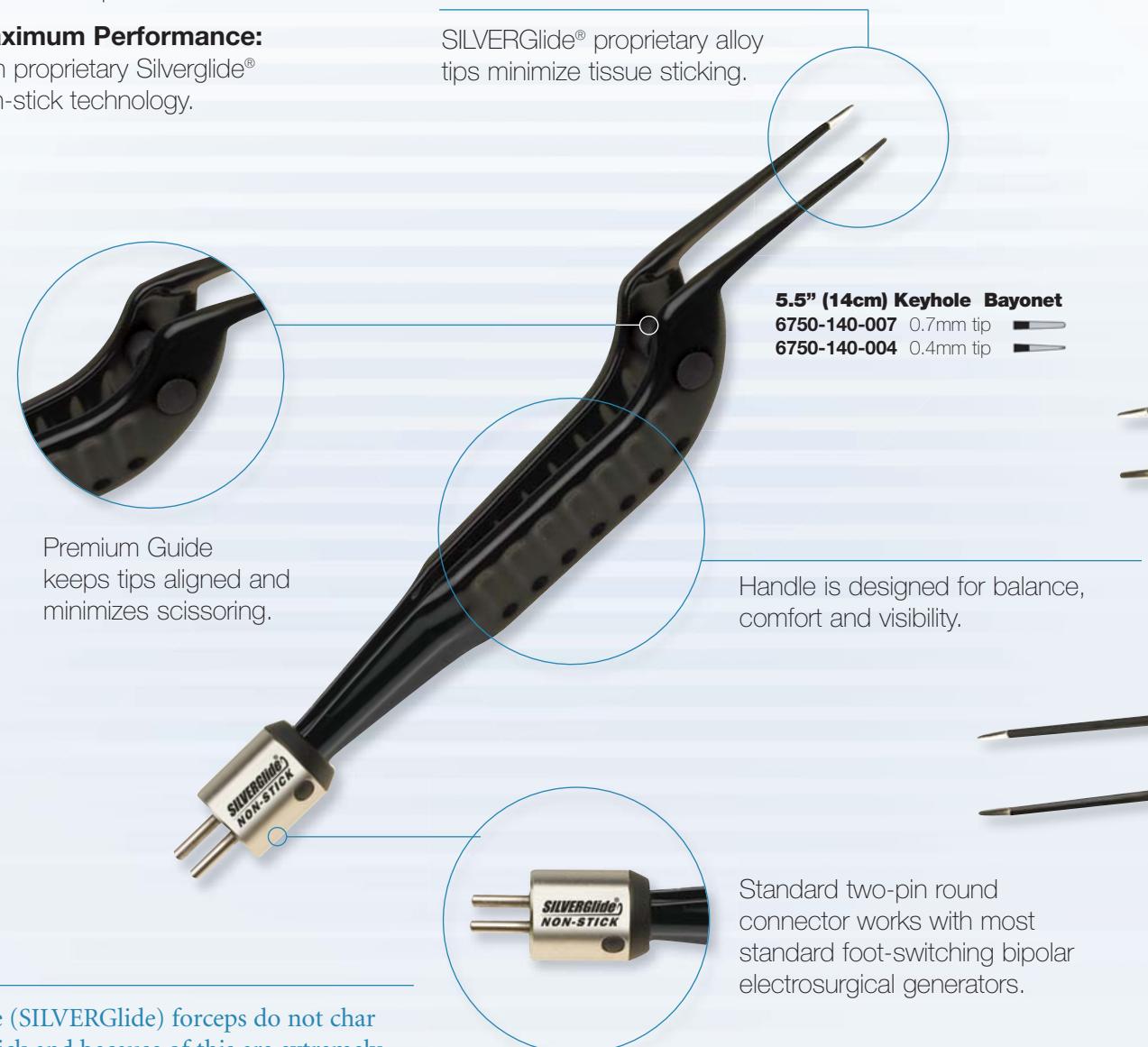

"The (SILVERGlide) forceps do not char or stick and because of this are extremely accurate in their application of energy. They have proven to be one of the best equipment purchases I have made."

— Mark Glasgold, M.D.,
Highland Park, NJ

SILVERGlide® Bipolar Forceps

SILVERGlide® Keyhole™ Maximum Access



SILVERGlide® Bipolar Forceps

SILVERGlide® Non-Stick Standard Bipolar Forceps




4.0" (10cm) Jewelers

6715-100-007 0.7mm tip
6715-100-004 0.4mm tip



5.0" (12.5cm) Adson

6715-125-012 1.2mm tip
6715-125-007 0.7mm tip
6715-125-004 0.4mm tip

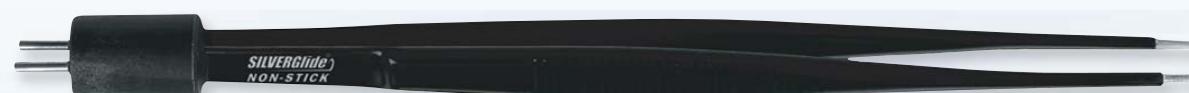

6.0" (15cm) Semkin

6715-150-012 1.2mm tip



7.5" (19cm) Cushing

6710-190-020 2.0mm tip
6710-190-015 1.5mm tip
6710-190-012 1.2mm tip

6710-190-010 1.0mm tip
6710-190-007 0.7mm tip
6710-190-004 0.4mm tip



8.3" (21cm) Cushing

6710-210-020 2.0mm tip
6710-210-015 1.5mm tip

6710-210-012 1.2mm tip
6710-210-007 0.7mm tip



7.8" (20cm) Scoville-Greenwood

6710-200-015 1.5mm tip

SILVERGlide® Accessories



Sterilization Trays:

- 6700-010-000 10 piece sterilization tray
- 6700-005-000 5 piece sterilization tray
- 6700-002-000 2 piece sterilization tray (forceps not included)



Other accessories:

- 6700-000-002 Polishing Cloths (10/pkg)
- 6700-000-001 Tip Guards (100/pkg)



6702-000-000 Disposable Cable: (50/pkg)

- Unique SnugShield feature facilitates a secure electrical connection
- Sturdy PVC rubber wire

6701-000-000 Reusable Cable:

- Sturdy silicon rubber wire
- Unique SnugShield feature facilitates a secure electrical connection

SILVERGlide® Bipolar Forceps

SILVERGlide® Non-Stick Guidestop Bipolar Forceps

The Guidestop on SP models keeps tips aligned, minimizes scissoring and aids in overall dissection.



9.7" (24cm) SP Bayonet

6720-240-020 2.0mm tip
6720-240-015 1.5mm tip
6720-240-012 1.2mm tip
6720-240-007 0.7mm tip
6720-240-004 0.4mm tip



8.7" (22cm) SP Bayonet

6720-220-020 2.0mm tip
6720-220-015 1.5mm tip
6720-220-012 1.2mm tip
6720-220-007 0.7mm tip
6720-220-004 0.4mm tip



7.5" (19cm) SP Bayonet

6720-190-020 2.0mm tip
6720-190-015 1.5mm tip
6720-190-010 1.0mm tip
6720-190-007 0.7mm tip
6720-190-004 0.4mm tip
6720-190-512 1.2mm tip, (angled up)
6720-190-612 1.2mm tip, (angled down)



7.0" (18cm) SP Straight

6725-180-420 2.0mm angled tip
6725-180-010 1.0mm tip
6725-180-007 0.7mm tip
6725-180-410 1.0mm angled tip



6.1" (15.5cm) SP Bayonet

6720-155-015 1.5mm tip
6720-155-012 1.2mm tip
6720-155-007 0.7mm tip
6720-155-004 0.4mm tip

SILVERGlide® Bipolar Forceps

SILVERGlide® Non-Stick Irrigating



Irrigation where
you want it...
not where you don't

Guide keeps tips aligned,
minimizes scissoring and
aids in overall dissection.

Separate fluid and electrical
connectors accept a greater
variety of cord-tubing sets.

Hollow tine design
brings irrigation to
the tip of the forceps.

10.2" (25cm) Bayonet
6740-250-012 1.2 mm tip
6740-250-007 0.7mm tip

8.7" (22cm) Bayonet
6740-220-012 1.2mm tip
6740-220-007 0.7mm tip

7.5" (19cm) Bayonet
6740-190-012 1.2mm tip
6740-190-007 0.7mm tip

"The SILVERGlide forceps are
remarkable. We use them in surgery
after surgery without tissue sticking."


— Gregory Keller,
M.D., F.A.C.S.
Associate Professor, UCLA,
Santa Barbara, CA

"The (SILVERGlide) forceps I purchased are really
wonderful. They don't stick, nothing sticks to
them, and they fare fine through the autoclave."

— Sorin Eremia, M.D.,
Los Angeles, CA
Associate Clinical Professor
Cosmetic Surgery, UCLA

SILVERGlide® Bipolar Forceps

What makes SILVERGlide® non-stick bipolar forceps so unique?

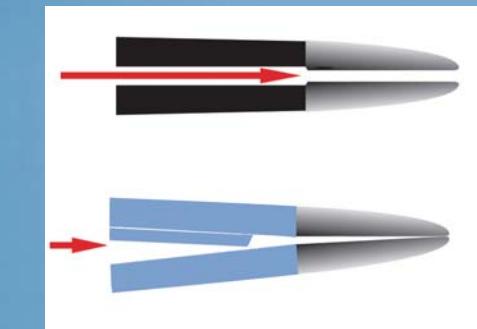

Platform tips facilitate
better visibility

The key uniqueness of SILVERGlide® is the optimal elimination of tissue buildup and sticking to the forceps tips. This reduces the need to stop and scrape the tips clean and allows the surgeon to work uninterrupted, focusing on the surgical procedure. A quick wipe with a moistened surgical sponge during the procedure keeps the tips clean and ready to use. Because the forceps tips remain clean, lower power settings may be used to achieve the desired surgical effect.

How does the SILVERGlide® Technology work?

SILVERGlide® is a proprietary technology that reduces the molecular bonding force and eliminates hot spots on the forceps tips. In fact, the level of thermal diffusivity of the SilverGlide® alloy is 37 times higher than that of stainless steel commonly used in standard forceps. Laboratory testing and field experience have shown that the non-stick characteristics of SILVERGlide® forceps are as good after multiple uses as they were the first time they were used.

How SILVERGlide® eliminates sticking




Standard forceps

"Hot Spots" from
sparks occur
during coagulation

The "Hot Spots"
cause localized
sticking

Tissue seared to
the forceps tips
is ripped away

Silverglide® non-stick

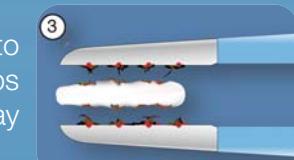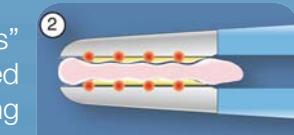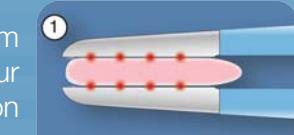

"Hot Spots" start
to occur during
coagulation

Heat is diffused
rapidly to prevent
sticking

The forceps can be
removed without
tissue sticking



access

to better Outcomes



The information presented in this brochure is intended to demonstrate the breadth of Stryker product offerings. Always refer to the package insert, product label and/or user instructions before using any Stryker product. Products may not be available in all markets. Product availability is subject to the regulatory or medical practices that govern individual markets. Please contact your Stryker representative if you have questions about the availability of Stryker products in your area.

Products referenced with TM designation
are trademarks of Stryker.

Products referenced with [®] designation
are registered trademarks of Stryker.

Literature Number: **9100-001-031 Rev. B**
UnDe/P.S.

Copyright © 2009 Stryker
Printed in USA

Stryker[®] Neuro Spine ENT
4100 East Milham Avenue
Kalamazoo, MI 49001 USA
t: 269 323 7700 f: 269 353 3026
toll free: 800 253 3210

www.stryker.com